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BASIC UDI- DI 859419979TUBES2U

Model:

- Suction Connecting Tube CH25
- Suction Connecting Tube CH30

Device description:

Size of tube:

- Ch 25 length 0.3 – 6.0 m
- Ch 30 length 0.3 – 6.0 m

Tube and Tips combinations:

- Funnel / Funnel
- Funnel / Fingertip
- Funnel / Tube part
- Funnel / Fingertip free
- Adapted Funnel / Fingertip free
- Adapted Funnel / Adapted Funnel, Fingertip slid-on
- Adapted Funnel / Adapted Funnel, Fingertip free
- Funnel / free end
- Fingertip / free end
- Free end / Free end
- Tube part / Tube part

Suction connecting tubes are disposable medical devices. Suction connecting tubes consist of a suction tube with or without connectors. The suction connecting tube is flexible and is made of transparent light blue PVC. The connectors (Funnel) are made of blue PVC, transparent Fingertip or transparent Tube part. We use phthalate-free material. The connectors are glued, pushed on or loosely attached to the connecting tube.

Depending on the specific product variant, the suction connecting tube can be used for various combinations of connections between the suction tip, suction catheter and suction container. The variant tube termination also allows connections between tubes, to suction tips, catheters or to a suction container.

Suction connecting tubes with connectors should be connected to the suction tip (for example Yankauer suction tip) or to a suction catheter, and the opposite end of the suction connecting tube should be connected to a suction container.

Connection and disconnection:

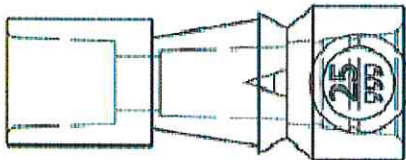
The connection is always via a flexible overlap between the tips or between the tips and the tube. The connection at operating vacuum is effected by the connecting force caused by the vacuum in the tube and thus does not come loose spontaneously. Disconnection after the suction is finished can be done by applying light axial pressure.

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**Connecting dimensions of the tips:**


For the funnel tip Ch25 and funnel tip Ch30, the connecting dimensions are the same for both types of tips. It is given by the internal conical connection:

- Small diameter  $\Phi$  7.2 mm
- Large diameter  $\Phi$  11.35 mm
- Length of the connecting tip 13 mm

For the Fingertip end, the connecting dimensions are given by the external graduated conical connection. External conical connection:

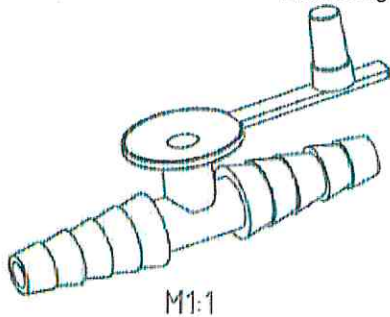- Small diameter  $\Phi$  5.3 mm
- Large diameter  $\Phi$  10 mm
- Length of the connecting tip 20 mm
- Diameter graduation:  $\Phi$  5.3 mm;  $\Phi$  6.4 mm;  $\Phi$  7.4 mm;  $\Phi$  8.0 mm;  $\Phi$  9.0 mm;  $\Phi$  10.0 mm

For a tube without a tip or for a tube where the tube connection is in the shape of a tube, the connecting dimensions are given by the size of the inner hole in the tube, taking into account the necessary overlap.

**Combinations for connection with connectors of connecting means:**

Connector:

Connection end or tip

- CH25 connector

- o Tube without CH25 connector
- o Tube without CH30 connector
- o Yankauer connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- o Suction container connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- o Suction tube  $\Phi$  6.0 mm –  $\Phi$  10.0 mm

- CH30 connector

- o Tube without CH25 connector
- o Tube without CH30 connector
- o Yankauer connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- o Suction container connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm

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- o Suction tube  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- Fingertip connector
  - o Tube Ch 25 without connector
- Tube part CH30
  - o Tube with CH25 connector
  - o Tube with CH30 connector
  - o Suction container connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- Free end of tube Ch25
  - o Free end of Tube Ch25
  - o Tube with connector CH25
  - o Tube with connector CH30
  - o Suction container connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
- Free end of tube Ch30
  - o Connector Ch25
  - o Connector CH30
  - o Yankauer connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
  - o Suction container connector  $\Phi$  6.0 mm –  $\Phi$  10.0 mm
  - o Suction tube  $\Phi$  6.0 mm –  $\Phi$  10.0 mm

The identical tubes are available in two sizes Ch25 and Ch30 in lengths from 0.3 to 6m in two versions - sterile and non-sterile. The device is suitable for use in MRI.

The sterile version of product and its blister packaging are sterilised with ethylene oxide (EO).

Products supplied non-sterile are intended for use in sets and kits, and are supplied without blister packaging. Sterilisation with EO is performed after they are packed into a set. In this case, sterilisation is not included in our delivery.

**Intended use:**

Suction and transport of body fluids in surgery

**Clinical benefit:**

The devices do not have a direct clinical benefit. The indirect clinical benefit is in conjunction with the suction tip. They help achieve the intended purpose of the devices with which they are used.

**Indications:**

Not specified

**Contraindications:**

Not specified

**Undesirable side effects:**

Not associated with the use of the product

**Intended patient population:**

Global population of all age groups, including young people.

**Intended users:**

Surgeons and nurses that are qualified and familiar with the surgical techniques.

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#### Warnings:

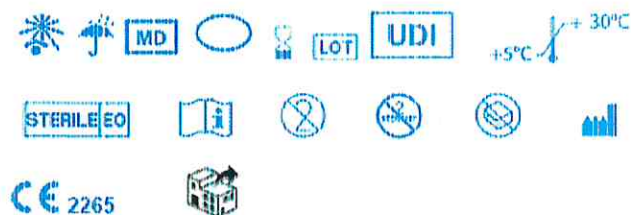
- The medical device is for single use. It is intended for one patient only. Reuse of a single-use device creates a potential risk of infection for the patient or user.
- Use aseptic techniques.
- Prior to each use, inspect the package. Do not use if the package has been opened or damaged.
- This medical device should not be used after the end of the shelf life shown on the packaging.
- Do not use a damaged or incomplete product.
- Before use, check the completeness and undamaged state of the medical device.
- The medical device is not intended for the removal of body fluids or storage for subsequent return to the human body.

#### Storage

- Store in the original packaging in a clean, dry, ventilated area away from direct light.
- To be stored in its packaging at temperatures between +5°C and +30°C.
- The storage temperature should not exceed +40°C for a long time (max. 30 days).
- During transport, the temperature should not be less than -20°C and should not exceed +50°C.

**Waste disposal:** After use, place in a hazardous bio-waste collection container. Dispose of as hazardous bio-waste in accordance with waste management regulations.

#### Symbols used on labelling:

**Note:** The user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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